ACCURYN® MAINTENANCE KIT USER GUIDE

Please follow these instructions as a quick guide to use the Accuryn Maintenance Kit.

For detailed instructions on using the Accuryn Maintenance Kit, please consult the Accuryn Monitoring System Operator's Manual. It is recommended to perform this maintenance at least once every three months.

Please call +1 833 ACCURYN (1 833 222-8796) for device related questions or problems

IFU-06-101233 Rev. B

Check Monitor Temperature

- Plug the Temperature Tool into the Temperature Port on the Monitor.
- Read the temperature displayed on the Monitor.
- Verify that the temperature displayed on the Monitor is within the range on the Temperature Tool (39.1±0.3°C/102.3±0.5°F).

Pin Cleaning

- Insert the Nylon Brush into Monitor
 Pins (M12 in picture below) once per pin.
- Only insert the Nylon Brush to the end of the bristles.
- Remove Nylon Brush from the Pins and clean bristles between each use.

